



Geistlich TauroSept®

User Guide


Step 1
Flush the catheter with Saline

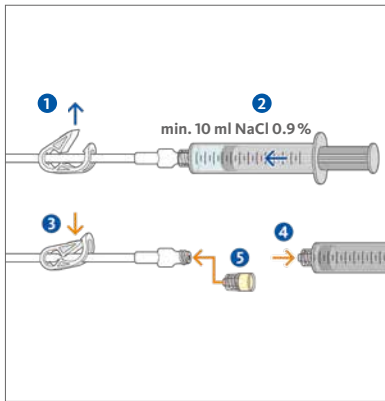


Step 2
Inspect TauroSept® vials for particles

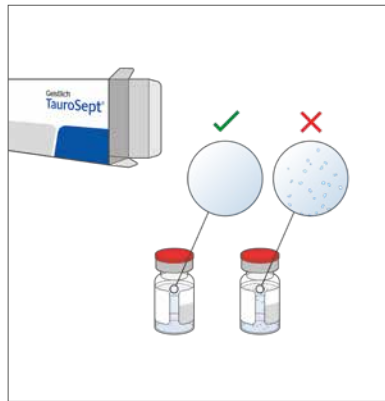


Step 3
Open lid and disinfect the septum

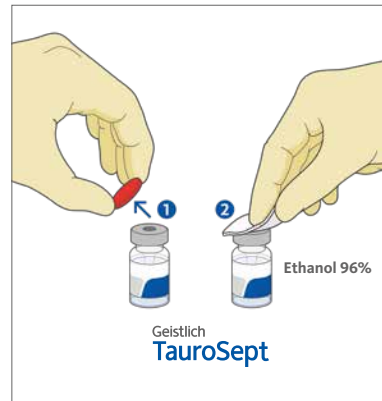


Step 4
Transfer required volume of TauroSept® into a syringe

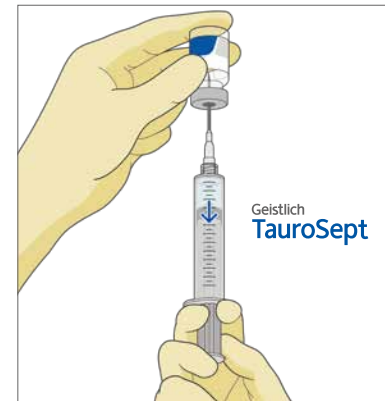


Step 5
Remove the syringe and needle

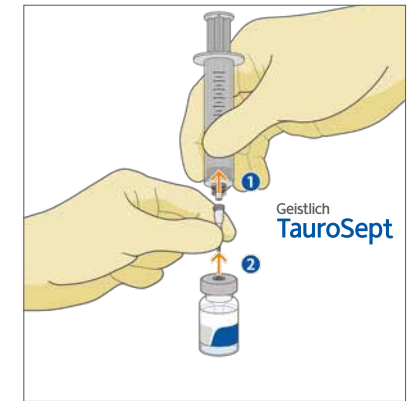


Step 6
Slowly fill TauroSept® into catheter

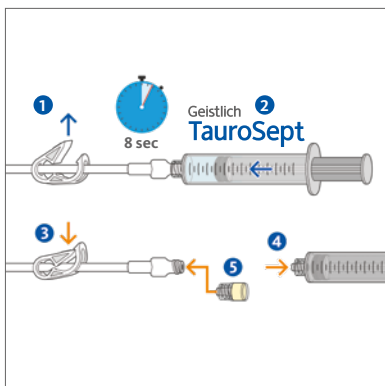


Step 7
Incubate TauroSept® 30 min to 3 weeks

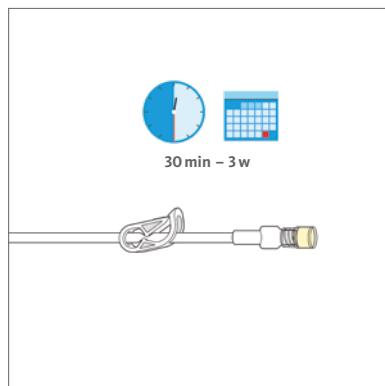

Step 8
Optional: Withdraw TauroSept® with syringe

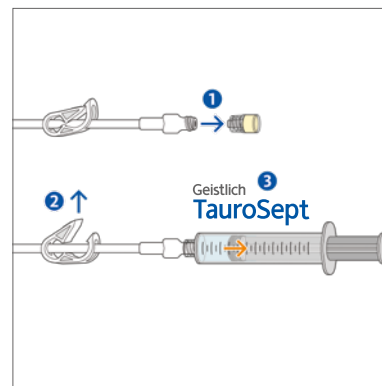

Step 9
Flush the catheter with Saline

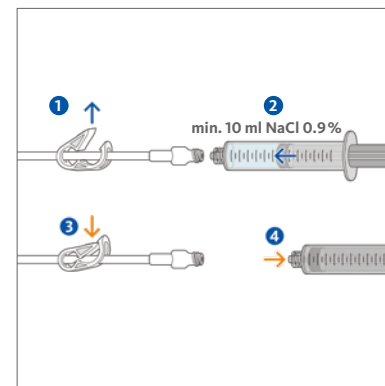

Step 10
Catheter usage

Catheter is ready to use e.g. for application of nutrition solution or hemodialysis

▶ Geistlich TauroSept®

User Guide

Composition

Active ingredient:
Taurolidine 2% Excipients:
Polyvinylpyrrolidone 5%
(PVP-17); water for injection,
traces of hydrochloric acid
or sodium hydroxide to
permit pH adjustment to 7.3

Storage

TauroSept® should be stored
in a **horizontal position** at a
controlled temperature of **15 °C
to 25 °C. Do not store in the
refrigerator.**

Shelf life

Do not use after the **indicated
expiry date.**

Usage

The product is sterile. The
content of a vial is for a **single
patient** only and must be used
within **48 hours** after the first
puncture. Write time and date

the vial was opened on the
vial label. **Re-sterilisation is
not possible!** Dispose of
unused portions of the solution
and withdraw residues from
access systems. **Do not use if
the packaging is damaged.**

Warnings

TauroSept® is not to be used for
systemic injection. The solution
should be **withdrawn** from the
catheter before next use. If
withdrawal of TauroSept® is not
possible for **technical reasons**,
e.g. with totally implantable
venous access systems (porta-
caths), or **clinically not wanted**,
e.g. in parenteral nutrition,
the **flushing** of TauroSept® can
be done without systemic effect.

Data on the use of the product
during pregnancy or lactation are
not available. For safety reasons:
women who are **pregnant or
breastfeeding should not be
treated** with TauroSept®.
The safety and efficacy of

TauroSept® have **not been in-
vestigated in children** before
skeletal maturity.

Catheter types

TauroSept® can be used with
any vascular access device.
Follow the manufacturer's
instructions for the particular
catheter utilized. Specific
catheter volumes are associated
with each device and must
be strictly followed.

For further information please
read the IFU.



Manufacturer

Geistlich Pharma AG
Business Unit Medical
Bahnhofstrasse 40
6110 Wolhusen, Switzerland
Phone +41 41 492 56 25
Fax +41 41 492 67 14
taurosept@geistlich.ch
www.geistlich-medical.com